

## Centricity RA1000 Workstation

### *Quick Reference Guide for Bookmarking*

#### Display an Exam

##### Setup Display:

- Select **Monitor Regions** for each Monitor
- Select the **Image Layout** for each monitor or virtual monitor
- **Link** similar planes as desired (Optional)
- **Cross-Reference** intersecting planes as desired (Optional)
- Select **Multi-Monitor Sheet** as desired (Optional)
- Select the **Exam** and **Series** to display in each Monitor Region (multi-modalities may be selected)
- Display specific image of interest per series region, if applicable
- Adjust **Window/Level** (Optional)
- Adjust **Zoom, Orientation, Invert** (Optional)

##### Saving a Bookmark:

- From the **Display Function** Menu, select **Save Bookmark**
- **Name** the Bookmark.

##### Applying a Saved Bookmark:

- Open an exam
- Choose **Bookmarks** from the **Display Function** Menu or Right Click in title bar
- **Select a Bookmark** to apply from the displayed list
- Select **Apply**



Depending on user preferences, the bookmark may be chosen as the default display. See user preferences in the Master Trainer Guidebook for details.




Suggested uses for bookmarks: Resident reading reviews with attending radiologist, pre-op set up for operating room display, conference or consulting presentations.

### Information Not Stored in a Bookmark Exam:


- User Overlays (ROI's, Text, Spine Labels, Measurements)
- Significant Images

### Accessing a Created Bookmark to be Applied for the Displayed Exam

- Open an exam
- Choose **More Bookmarks** from the **Display Function** Menu
- Choose a **Bookmark** from the list
- Click **Apply**

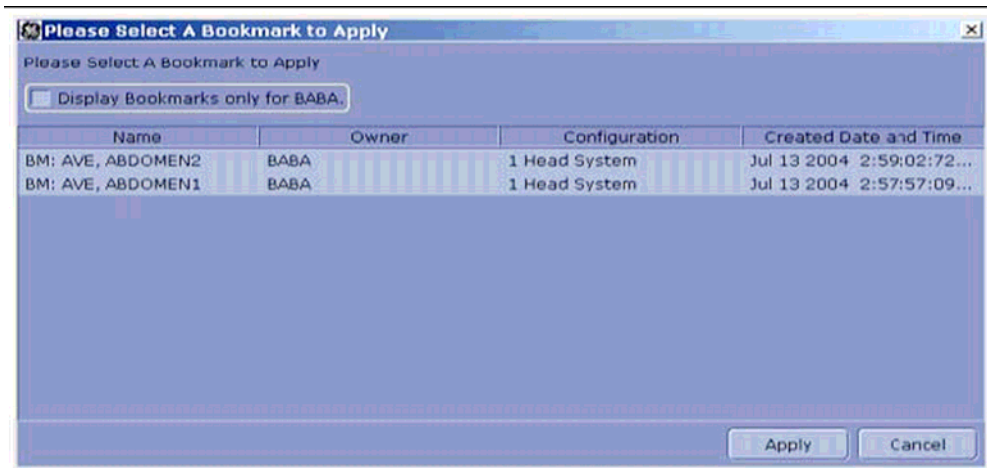

### Setting User Preference for Exam Opening Options (Utilities Palette)

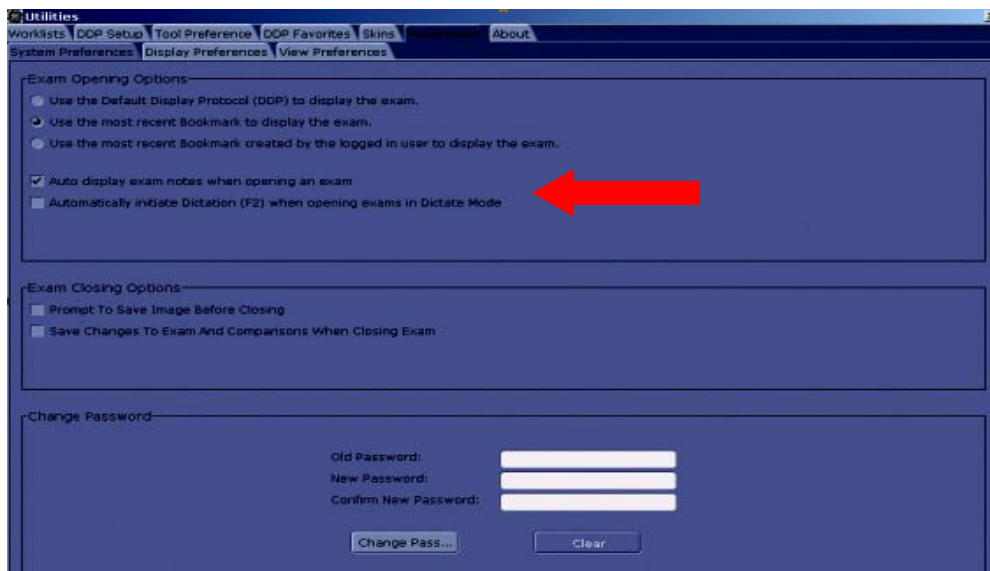

### Trouble

- Call your facility's Help Desk or contact your System Administrator.