

GE Healthcare



## Centricity RA1000 Workstation

### *Quick Reference Guide for Clinician Workflow*

#### Log In

- Type in User ID & Password and press [Enter] on the keyboard.

#### Find a Patient and Select an Exam

- Type a patient identifier in the box under the *appropriate* heading (i.e. Patient Name, Patient ID, etc.) in the Work Mode palette.
- To select a specific exam from the list, single-click on the exam so that it is highlighted.

#### Display an Exam

- **Double click** on the desired exam or L click the **Display Exam** icon.

A rectangular button with the text "Display Exams" in a bold, sans-serif font.

- Exam will display on monitor(s).

#### Close Exam

- Click on the **Close Exam Icon** at the upper right corner of the screen.



#### Log Out

- Click on the **Logout Icon** in the lower right corner of the screen.



#### Trouble

- Call your facility's Help Desk or contact your System Administrator.

Three palettes of information will be displayed in the default view: **Work Mode** (all patients), **Patient Jacket** (displays prior studies for the patient that is highlighted in the **Work Mode** palette) and the **Report** (report of the patient exam highlighted will show demographic information and the body of report.)

The screenshot displays the GE Healthcare software interface with three main palettes:

- Work Mode:** A table listing exams. A red arrow points to the first row. The table has columns: Patient Name, Patient ID, LastF..., Procedure, Md, Img, Study Time, Online, and Status.
 

Patient Name	Patient ID	LastF...	Procedure	Md	Img	Study Time	Online	Status
NR03262004	NR03262004	2004	Unspecified CT	CT	399	2004.03.27 08:50:...	Y	Arrived
NR03262004	NR03262004	2004	Unspecified MR 0	MR	200	2004.03.27 08:51:...	Y	Dictated
NR03262004	NR03262004	2004	Unspecified RF	RF	200	2004.03.27 08:52:...	Y	Verified
LAYFIELD, STEPHA...	00013913	3913	Unspecified US	US	29	2004.01.07 12:34:...	Y	Dictated
WHITE, CHARLES	583020	3020	Unspecified MR	MR	0	2004.04.07 23:10:...	N	Dictated
WHITE, CHARLES	583020	3020	Unspecified MR	MR	16	2004.04.09 16:35:...	Y	Dictated
WHITE, CHARLES	583020	3020	Unspecified MR	MR	32	2004.04.09 15:53:...	Y	Arrived
WHITE, CHARLES	583020	3020	CT Procedure 1	CT	0	2004.04.09 15:53:...	N	Ordered
STANDARD_CT EXA...	EF033004-16	4-16	Unspecified CT	CT	311	2004.03.31 10:35:...	Y	Read Offline
DAC FCE TEST 2	EF040104-FCE-02	E-02	Unspecified MR 0	MR	1	2004.04.01 03:36:...	Y	Verified
- Patient Jacket:** A table showing historical exams for the selected patient. A red arrow points to the 'Display Ex...' button.
 

Patient Name	Patient ID	Procedure	Md	Img	Study Time	Online
NR03262004	NR03262004	Unspecified CT ABDOMEN	CT	200	2004.03.27 08:51:44	Y
NR03262004	NR03262004	Unspecified MR 0	MR	200	2004.03.27 08:51:12	Y
NR03262004	NR03262004	Unspecified CT	CT	399	2004.03.27 08:50:45	Y
- Report:** A form displaying patient and exam details. A red arrow points to the 'Display Ex...' button.
 

Patient Name: BARON, SHELBY MM  
 MRN: 46324  
 Reason for Exam: Head traumaLOCMVA  
 Sex: F  
 Pat Location: Unspecified  
 Pat Residency: Unknown

Study Date/Time: Jul 29, 2004 08:55:53  
 Birth Date: Sep 15, 1971 (Age: 32Yrs)  
 Order Comments: N/A  
 Department: CT  
 Ward/Clinic: Unspecified  
 Priority: Routine

Procedure: CT HEAD CONTRAST  
 Status: Ordered  
 Physician: Unspecified  
 Order No: N/A  
 Exam ID: N/A  
 Sched Date/Time: Jul 29, 2004 09:04:26

Standard Reports: No Repo...  
 Report Approval: Select Ap... Set Exam Status: Mark Dict... Edit Print

Once the report is available it will display in this area