





## Centricity RA1000 Workstation

### [Quick Reference Guide Common Keyboard Shortcuts](#)

<b>Alt-D</b>		Display images
<b>Alt-O</b>		Log-out of Centricity Workstation
<b>F1</b>		Access on-line help
<b>F2</b>		Start dictation (opens dictation palette with integrated system)
<b>F3</b>		Displays the saved My Desktop Workstation Palettes
<b>F6</b>		Mark exam dictated
<b>F7</b>		Displays GE Default Workstation palettes (Work Mode, Patient Jacket and Report Palette)
<b>F9</b>		Hides all the Workstation palettes
<b>F12</b>		Approve and Complete the exam
<b>Alt + I</b>		Displays image overlay information
<b>Ctrl-U</b>		Toggles the Utilities palette opened and closed
<b>Ctrl-W</b>		Toggles the Work Modes palette opened and closed
<b>Ctrl-Alt-N</b>		Create Exam Note
<b>Ctrl-Alt -V</b>		View Exam Note(s)
<b>Alt-W</b>		Close current exam
<b>Ctrl-Alt-S</b>		Save Current and Comparison exams
<b>Ctrl-Alt-W</b>		Close all open exams
<b>Ctrl +</b>		Changes size of magic glass region
<b>Alt +</b>		Changes Window/Level setting of magic glass image
<b>Shift +</b>		Changes the magnification ratio
<b>Ctrl - Alt +</b>		Invert grayscale of magic glass image
<b>Left Arrow</b>		Displays the previous single image in a series
<b>Right Arrow</b>		Displays the next single image in a series
<b>Up Arrow</b>		Display the previous series in exam
<b>Down Arrow</b>		Displays the next series in exam