



Centricity RA1000 Workstation

Quick Reference Guide Default Display Protocol (DDP)

Creating a DDP

- Open an exam.
- **Display** images as desired (set up monitor regions, select the series you wish to display in each monitor, choose format for each monitor region, add desired connect features such as cross-reference, link, and/or multi-monitor sheet).
- Click **Display Functions** drop down menu.
- Select **Save**. You may have to expand your Display Functions menu to see this option.
- DDP dialog box opens with default box checked and the open exam automatically checked under exam list.
- Select any other **Body Parts** and associated **Exams** for use with this DDP.
- Check the box in front of **Window/Level**, **Magnification** and **Image Orientation** as desired. (Optional)
- Click **Save**.



Window/Level, Magnification and Image Orientation will display as it is sent by the Modality (or last saved value) unless you wish a consistent Window/Level, Magnification or Image Orientation to be applied each time these exams are displayed.



The “Force DDP tool Parameters by default” setting in Utilities Palette under display preferences must be checked if you wish to save a Default Display Protocol (DDP) with window/level, magnification and/or image orientation changes.



“Use the Default Display Protocol to display the exam” setting in Utilities Palette under system preferences must be checked if you wish to use you DDP to display images vs. displaying Bookmarks for this study.

Copying a DDP

- Display an exam
- Click **Display Functions** drop down menu
- Select **More Display Protocols**
- Select the **Name** of another user whose DDP you wish to copy
- Select DDP to be copied from the user's list
- Click **Apply**
- Verify that the DDP as shown on the monitor(s) is the desired display you wish to copy for the displayed exam.
- Click **Display Functions** drop down menu.
- Click **Save** and follow the steps listed above to save as your DDP.

Deleting a DDP

- Open an exam.
- Click **Display Functions** drop down menu.
- Select **Delete**
- A list of the user's DDPs will appear.
- **Select the DDP** in the list you wish to delete.
- Click **Delete**.

Trouble

- Call your facility's Help Desk or contact your System Administrator

