



## **Centricity RA1000 Workstation**

### ***Quick Reference Guide for Exam Functions***

#### **Save Current Exam**

- Used to save changes made to an exam

#### **Save Current + Comparisons**

- Used to save changes made to both current and comparison exams that are open

#### **Close Current + Comparisons**

- Used to close both current and comparison exams

#### **Close All Open Exams**

- Used to close all open exams including those open in the background

#### **Show Exam As Acquired**

- Used to removes all post acquisition image manipulations from the displayed images

#### **View Sig Images**

- Used to display the only the significant images in the exam.

#### **Send Exam**

- See Quick Reference Guide for Send Exam Option

### **Print Images (Optional)**

- Select **Print Images** from drop down menu
- Print dialog box appears
- Choose camera from drop down menu
- Choose format layout from drop down menu
- Choose selected images, significant images or all images to print
- Print scout and series (optional). Images must be cross-referenced first
- Choose number of copies to be printed, set priority, and choose film orientation
- Note number of films to be printed
- Click on **Print**. Films will print to camera selected above



The **Actual Size** option in the format menu is used for CR exams.

Make sure your film format layout is within your printer specifications.

Make sure to check the number of films to be printed with the selected format before clicking the print icon.

### **Mark Unverified, Verified or Dictated**

- Allows user to change exam status to unverified, verified or dictated. Status changes are privileged, so not every user has this capability.

### **Send Exam to Remote WS (workstation)**

- Allows user to send images (using DICOM send) to a remote workstation such as a radiologist workstation located in their home.

### **Start Dictation (Optional)**

- Option for interfaced dictation system only. Allows user to launch dictation module.

### **Save for Conference/Delete Conference**

- Allows user to save or delete a Conference Display Protocol (CDP).

### **Trouble**

Call your facility's Help Desk or contact your System Administrator