



Centricity RA1000 Workstation

Quick Reference Guide for Image Functions

Select All and Deselect All

- Allows user to select or deselect all images in a series.

Reject Selected Images

- Allows user to reject selected images in a series. A dialog box will display the number of images to be rejected and require a reason for rejection.
- Click **OK**
- Once images are rejected from Centricity, they may not be resent without modifying the DICOM header at the modality.



Verify the number of images selected for rejection before clicking OK.

Add or Remove Image From Folder

- Allows user to add or remove images from a manual folder.
- Display contents of folder and select **Remove Images From Folder** option.
- Select **Add Images To Folder** and select manual folder.

Image Overlay

- Displays modality information such as slice thickness or table location on the image.

Overlay Graphics

- Displays any overlay information that was saved as a graphic file when the exam was acquired.

ROI/Annotation

- Allows user to hide/show ROI and Annotations applied to images from the Centricity workstation.

X-Ref Lines

- Used for **stack** format, **cross-referenced images**.
- Displays cross-reference lines for all images in a series.
- Single line for image displayed will appear when initially cross-referenced. Lines will hide when monitor is active if user has selected option in utilities palette.

Token Scout

- Used for **sheet** format, **cross-referenced images**.
- Allows user to view Token Scout with cross-reference line in lower left corner of each image in a series.

Picture in Picture

- Allows user to display a small version of the entire image in the upper-left corner of the image.
- A white rectangle box indicates which area of the image is currently being viewed.

Spine Labeling

- See Master Trainer Guidebook or Quick Reference Guide for Spine Labeling.

Select a Scout

- Used for **sheet** format, **cross-referenced images**.
- Click on the image user desires as token scout. The new scout is immediately displayed in the lower left corner of the display area.

Change Annotation Font

- Select **Change Annotation Font**. A Change Annotation Font dialogue box will open.
- Select the **Font**, **Type** and **Size** from the drop-down menus. Click **OK** to apply changes.
- The change will only apply to the opened exam. The system will return to user default setting in Utilities once the exam is closed.

Window Level Presets

- Allows user to select **Window Level Presets**. A sub-menu displays choices. Click on the W/L preset to apply to the image. Available presets are determined by the modality being viewed, and the user preference settings.

LUTS

- Allows user to select Lookup Tables (**LUTS**).
- A sub-menu displays choices.
- Click on the LUT preset to apply to the image. Used for Pet, Mammography and Nuclear Medicine modalities.

Image Filters

- Allows user to select **CT Image Filters**.
- A sub-menu displays choices.
- Click on the **Image Filter** preset to apply to the image. Image filters should only be applied to CT exams produced on a GE scanner. Filters may not be saved.