



Centricity RA1000 Workstation

Quick Reference Guide for Viewing and Saving Presentation States

PRESENTATION STATES

The Consistent Presentation of Images (CPI) feature maintains the consistency of presentation of grayscale images, and their presentation state information.

The following operations can be saved in the presentation states:

- Annotate an image
- Apply a zoom factor
- Adjust window and level
- Modify the shutters
- Image Mask
- Pan

The following operations cannot be saved in the presentation states:

- Magic Glass
- Pixel Lens
- Spatial Cursor / Multiplanar Localization
- Significant Image

VIEWING PRESENTATION STATE

1. From either the **Work Modes** or **Patient Jacket** palette, open an exam.
2. In the **Display Functions** menu, click **Presentation State Functions** (collapsed menu). If the **Display Functions** menu is expanded, the list of Presentation States will automatically appear.
3. From the Presentation States list, select the Presentation State that you want to apply.



The name of the applied Presentation State displays in the right side of the Function Bar.



The number of Presentation States that display in the list is configurable. If there are more Presentation States than can be displayed on the list, click **More Presentation States**. The entire list will display. Select the Presentation State to view and click **Apply**.

SAVING A PRESENTATION STATE MANUALLY

1. From either the **Work Modes** or **Patient Jacket** palette, open an exam.
2. Perform any operation that can be saved as Presentation State.
3. From the **Display Functions** drop down menu, select **Presentation State Functions** (collapsed menu) -or- **Manage Presentations** (expanded menu).
4. Click **Save Presentation State**.
5. The Save Presentation States dialog box will display.
6. In the **Presentation Label** text box, type the label.
7. In the **Description** text box, type the description.
8. Click **Save**.

SAVING A PRESENTATION STATE WHEN MARKING EXAM DICTATED

1. In the **Utilities** palette, make sure the **Preferences, Display Preferences, Display Options** checkbox for **Automatically save PSG when marked dictated** is selected.
2. From either the **Work Modes** or **Patient Jacket** palette, open an exam.
3. Perform any operation that can be saved as Presentation State.
4. When dictation is complete, mark the exam dictated.

SAVING A PRESENTATION STATE WITH KEY IMAGE NOTES

1. With an exam open, select the **Key Image Selection** tool.
2. Click on the desired image(s) - The selected image displays the Key Selected Image Icon.
3. Select **Notes** from the Context Menu -or- hover the cursor over the **Key Selected Image** Icon and click the **Save Key Image Note** pop-up.
4. Select a **Title, Subtitle** (if available), and enter a free text note (if desired).
5. Select **Include Current Presentation**.
6. **Save** note.

TROUBLE

Call your facility's Help Desk or contact your System Administrator